



⚠ WARNING: Due to sterilization, this product can expose you to Ethylene Oxide, known to the State of California to cause cancer and/or birth defects or other reproductive harm. For more information, go to www.P65Warnings.ca.gov



USREP Recibio, Inc.
1860 FM 359 #171, Richmond, TX 77406
Phone: 1-844-798-4846
www.cealogic.com

 Foshan United Medical Technologies Ltd.
No. 1 Huixing Road, Yanbu, Dali, Nanhai, Foshan, Guangdong
Province, 528247, China www.umatmedical.com

 **CeaLogic™ FILM**
Transparent PU Film Dressing - Frame Style

REF NO. 9357



Product Description

CeaLogic™ Film Dressing consists of a thin film backing with a non-latex, hypoallergenic Adhesive. The dressing is transparent, allowing continual site observation. It is waterproof and impermeable to liquids and bacteria. CeaLogic™ Film Dressing is individually packed and terminally sterilized.

Patient Target Group

Any patient who presents with an appropriate wound type.

Indications

CeaLogic™ Film Dressing can be used as a securement for IV catheter or wound dressing, to cover and protect catheter sites and wounds, as a secondary dressing, as a protective cover over at-risk skin, to secure other devices to the skin. Do not use the dressing as a replacement for sutures and other primary wound closure methods.

Warnings

- CeaLogic™ Film Dressing should not be placed over infected wounds.
- Do not insert the needle through the dressing
- Ensure patient is not sensitive or allergic to adhesive products prior to application.
- If infection is suspected, remove the dressing, inspect the catheter site directly, and determine appropriate medical intervention.
- For external use only.
- Do not use if individual pouch is open or damaged.
- Do not reuse, reuse may cause serious infection.

Precautions

- Do not stretch the dressing during application as tension can cause skin trauma.
- Make sure the skin is clean, free of soap residue and lotion and allowed to dry thoroughly before applying the dressing to prevent skin irritation and to ensure good adhesion
- Choose a dressing size that will adequately cover the insertion site and the catheter. At each dressing change, pay careful attention to disinfecting the skin around and under the character.
- CeaLogic™ Film Dressing should not be re-sterilized.

Application

1. Prepare the wound site according to institution protocol. Allow all cleansing solutions and skin protectants to dry completely. Take CeaLogic™ Film Dressing out from the package.
2. Peel liner from dressing, exposing adhesive surface. NOTE: For CeaLogic™ Film Dressing Frame Style, remove and discard the latter liner before touching skin.
3. Center the transparent portion of the dressing over the site. Apply firm pressure to entire dressing starting in the center to the edges to enhance adhesion. NOTE: Remove the remaining liner of CeaLogic™ Film Dressing Frame Style and smooth the dressing from the center outward to enhance adhesion. Do not stretch the dressing during application.
4. For CeaLogic™ Film Dressing Frame Style, slowly remove frame while smoothing down dressing edges with your fingertips.
5. For CeaLogic™ Film Dressing Frame Style, slowly remove PE/PET film while smoothing down dressing edges with your fingertips.
6. Smooth the dressing from the center towards the edges, using firm pressure to enhance adhesion.
7. Record information on label, then remove it from liner and place on dressing.

Removal

1. Remove any tape strips applied to the top of the dressing.
2. Using a low and slow removal technique, start removing the dressing from corner. Avoid skin trauma by peeling the dressing back, rather than pulling it up from the skin.
3. Support the skin with your fingers while removing the dressing

Disposable

The dressing should be disposed of in accordance with local waste guidelines.

Storage

Protect from freezing, avoid excessive heat.

SYMBOLS USED ON LABELLING



Single use



Do not Re-sterilize



Use by: year and month.



Sterile. Method of sterilisation: EO



Product Reference



Batch number.



Attention, see instructions for use.



Does not contain Latex



Storage Conditions



Manufacturer



Representative



Date of manufacture



Keep Away from Sunlight



Keep Dry



Do not use if package is open or damaged



Medical Device



See instruction for use



Single sterile barrier system



Global Trade Item Number



Unique device identifier