



⚠ **WARNING:** Due to sterilization, this product can expose you to Ethylene Oxide, known to the State of California to cause cancer and/or birth defects or other reproductive harm. For more information, go to www.P65Warnings.ca.gov

 **CeaLogic™ RIBBON**
Chitosan Absorbent Gelling Fiber Dressing

REF NO. 9356



USREP Recibio, Inc.
1860 FM 359 #171, Richmond, TX 77406
Phone: 1-844-798-4846
www.cealogic.com

 Foshan United Medical Technologies Ltd.
No. 1 Huixing Road, Yanbu, Dali, Nanhai, Foshan, Guangdong
Province, 528247, China www.umtmedical.com



Product Description

CeaLogic™ Ribbon is a soft, sterile, absorbent fibre wound dressing made from Chitosan. Chitosan is known for its bacteriostatic and haemostatic properties.

CeaLogic™ Ribbon is highly absorbent, conformable and transforms into a soft gel upon absorption of wound exudate. CeaLogic™ Ribbon helps maintain a moist environment for optimal wound healing, conforming closely to the wound bed for optimal interaction. The dressing can aid autolytic debridement and will remain intact during use (one-piece removal). CeaLogic™ Ribbon will not damage newly formed tissue.

Indications

CeaLogic™ Ribbon is indicated for the management of moderate to heavily exuding chronic and acute wounds. CeaLogic™ Ribbon may be used to control minor bleeding in superficial wounds. CeaLogic™ Ribbon can be used on infected wounds or wounds in which there is an increased risk of infection under the supervision of medical professionals. Under medical supervision CeaLogic™ Ribbon may be used for the management of the following wounds:

- Pressure sores
- Venous ulcers
- Diabetic ulcers
- Leg ulcers
- Cavity wounds
- Donor sites and Graft sites
- Surgical wounds (e.g. post-operative, wounds left to heal by secondary intent)
- Skin abrasions and lacerations
- Superficial and partial thickness burns
- Trauma Wounds

Contra-indications

The CeaLogic™ Ribbon is not indicated for use on the following:

- Surgical implantation
- To control heavy bleeding
- Individuals who are sensitive, or who have had an allergic reaction, to the dressing.
- 3rd degree burns

Directions for use

Site Preparation

Debride when necessary and irrigate/clean the wound site in accordance with standard protocols.

Dressing Selection

For flat or shallow wounds select an appropriate size dressing that allows for a minimum of 0.5 cm overlap onto the skin surrounding the wound (to allow for the dressing to gel and to maximise conformability). Use a ribbon (rope) dressing for cavity wounds."

Dressing Application

1. Always use aseptic techniques when dressing a wound
2. Cut, fold or layer, the dressing to fit the wound
3. Loosely fill deep wounds, to allow the CeaLogic™ Ribbon to absorb wound fluid and to expand filling the wound. Do not pack tightly.
4. When using CeaLogic™ Ribbon ribbon dressings in deep cavity wounds, insert the ribbon in one piece and leave at least 2.5cm outside the wound for ease of removal
5. Cover and secure with an appropriate secondary dressing
6. For dry wounds (to aid a moist wound environment) place a CeaLogic™ Ribbon on the wound and wet with sterile fluid or saline over the wound area only. Cover the dressing with an occlusive or semi-occlusive dressing to maintain moisture at the wound site and to avoid subsequent dressing adherence to the wound
7. For minor bleeding wounds, apply the CeaLogic™ Ribbon to the wound with a light pressure, maintain the pressure for few minutes until the bleeding is stopped, secure the dressing with a suitable secondary dressing

Dressing Change and Removal

1. CeaLogic™ Ribbon shall be changed when the dressing has reached it's absorption capacity, or when the conditions of the wound or the surrounding skin or the level of exudate requires.
2. If simple removal of the dressing is difficult due to adherence, it should be fully saturated with sterile saline or sterile water to fully gel the dressing, remove slowly
3. Irrigate the wound site with a suitable wound cleanser (e.g. sterile saline) prior to application of a new dressing

Storage

Store in a cool, dry place.

Do not re-sterilise

The use by date of this product is printed on the packaging.

SYMBOLS USED ON LABELLING



Single use



Do not Re-sterilize



Use by: year and month.



Sterile. Method of sterilisation: EO



Product Reference



Batch number.



Attention, see instructions for use.



Does not contain Latex



Storage Conditions



Manufacturer



Representative



Date of manufacture



Keep Away from Sunlight



Keep Dry



Do not use if package is open or damaged



Medical Device



See instruction for use



Single sterile barrier system



Global Trade Item Number



Unique device identifier